**MONOFILAMENTO**  
**MONOFILAMENT**  
**MONOFILAMENT**  
**EINZELFADEN**  
**MONOFILAMENTO**  
**MONO-FILAMENTO**  
**MONOMHMATIO**  
**الخيط الوحيد**

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGS ANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**دليل الإستعمال والرعاية**

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**MONOFILAMENT**  
 Show the filament to the patient and touch it to his/her hand or arm so that he/she knows it does not hurt.

Use the 10 gram filament to test sensation at the indicated sites on each foot as shown. Apply the filament along the perimeter of, and NOT directly on, an ulcer, callus, scar or necrotic tissue.

Hold the filament perpendicular to the skin and use a smooth motion when testing. Use a 3-step sequence that includes (1) touch the skin, (2) bend the filament, and (3) lift from the skin (See Figures 1- 3). Donotuserapidmovement. The approach, skin contact, and departure of the filament should be approximately 1.5 seconds in duration.

Ask the patient to respond "yes" when the filament is felt. If the patient does not respond when you touch a given point on the foot, continue on to another site. When you have completed the sequence, REPEAT on the area(s) where the patient did not indicate feeling the filament.

Use the filament in a random sequence.

**LOSS OF PROTECTIVE SENSATION AT ANY ONE OF THE EIGHT SITES (shown on the diagram above) INDICATES A FOOT AT HIGH RISK.**

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