ISTALLAZIONE / INSTALLATION

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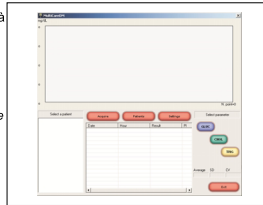
*Insert the CD in the reader, the mask "install" will appear.
Select USB DRIVER to allow the program to verify and install the needed drivers.
After last operation, select "Mcin" and follow the guided procedure.
After concluded the procedure, go out from the program of installation by select "EXIT".
To start the software on the main menu select "multicare memory reading" (default destination is C:/Program).*

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*Connect the USB cable to the PC.
Maintaining the instrument switched off, connect it to the other extremity of the USB cable.
When you switch on the instrument, icon "PC" will appear on the display.
Start the program as described above.*

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
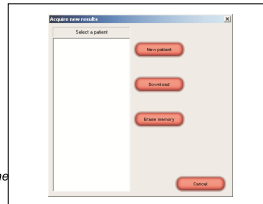
*The boton "Acquire" allows to download the data from the instrument.
Before to go on, the program ask to select or create patient ID.
When this operation is executed, if you clic on "download data", positioning the data in the file that was selected before.*

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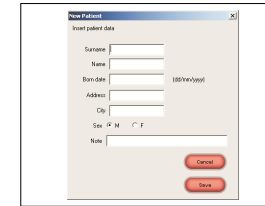
*Using this button, you can create new patient file.
See picture on side, example of anagraphic scheda.*

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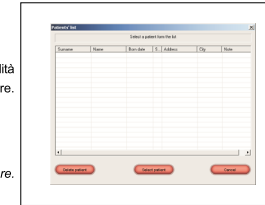*The boton "Patients" allows to enter in the mask to see and manage all the inserted data.
Selecting a patient, you can see specific data and you can modify the data, print the data, cancel the data and export them out the database of the software.*

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